

Date: 22/04/2015

Instruments: *Convergys*<sup>®</sup> X5

**ATTENTION- HIGH IMPORTANCE – PLEASE READ CAREFULLY!!**

**Subject:** *Convergys*<sup>®</sup> X5 software release v. 1.5.2361

**Priority:** as needed or next service call

**Effective Date:** 27.06.2016

Dear Partner,

We are pleased to inform you that *Convergys*<sup>®</sup> X5 software version 1.5.2361 has now been released.

Main features:

- Windows 8.1 software responding message will not appear in AS mode when the system is busy with too much data.
- Bug fixed on communication error - SW hanging in case of no HL7 receiver.
- Default screensaver time now set at 15 min.
- Win 8 password will not expire after 42 days.
- Communication problem fixed between High Level Software and Low Level Software. The root cause of the problem was the WIN8.1 channel allocation for the FTDI chip USB protocol.
- Input window header of PCT and MPV or PLCR and PLCC fixed when entering QC target values and ranges data.
- Instrument will now exit screensaver mode when pressing the START button.

If you have any questions please do not hesitate to contact us.

Best regards,

**Convergent Technologies GmbH & Co. KG**

**Ringstrasse 14, 35091 Coelbe, Germany**

**Tel: +49 6421 8869948**

**Fax: +49 6421 8869958**

---

**Convergent Technologies GmbH & Co. KG**

**Registered Office**

Germany

Ringstrasse 14, 35091 Cölbe, Germany

Tel: +49 6421 8869948 Fax: +49 6421 8869958

**Homepage:** [www.convergent-technologies.de](http://www.convergent-technologies.de)

**e-mail:** [info@convergent-technologies.de](mailto:info@convergent-technologies.de)

**Ust-Id:** DE 241291597

Steuernummer: 031 310 30121

Amtsgericht Marburg HRA 4125

**Geschäftsführer:** Umesh Bhalla

**Bank:** Volksbank Main-Tauber eG, Tauberbischofsheim,

**EURO Transfers**

Account No. 67131401

IBAN DE66 6739 0000 0067 1314 01

BIC GENODE61WTH

SWIFT GENODEFF

**USD Transfers**

Account No. 567131408

IBAN DE97 6739 0000 0567 1314 08

BIC GENODE61WTH

SWIFT GENODEFF

REV006